

STANDARD OPERATING PROCEDURES
DIVISION OF COMPARATIVE MEDICINE
UNIVERSITY OF SOUTH FLORIDA

SOP#: 1105.5

Date Issued: 01/00

Date Revised: 5/16

Page 1 of 3

TITLE:

Surgi-Vet® and Matrix®

V. MAINTENANCE

1. Inspect condition of unit and electrical cord/plug to ensure safe operation. Equipment determined to be unsafe will be removed from service immediately.
2. Clean unit and sensors prior to each use by wiping with a mild soap solution (e.g., dilute chlorhexidine 1oz. /1 gal. water) and soft cloth. Disinfect surfaces but not the screen) prior to survival procedures by wiping with isopropyl alcohol.
3. Maintain battery charge by connecting external charge to rear of unit
4. Moisture trap should be replaced when full and occludes.
5. Check pneumatic system for leaks periodically and after replacing moisture trap.
6. Replace CO

- g. When the “**Calibration Complete**” message appears, the unit is done. Disconnect the calibration test fixture and exit all menus.
4. Refer to manufacturer’s manual for additional information.
5. Calibration is documented on **CMDC 205, Anesthesia Machine Hang Tag** with the date of calibration.

VII. REFERENCES

1. Refer to manufacturer’s manual for additional information.

Approved:

Date: